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EN INSTRUCTIONS

WARNING:
TO PROTECT YOUR HEALTH AND THAT OF OTHERS, IT IS VERY IMPORTANT TO FOLLOW THESE INSTRUCTIONS.

- Wear this mask when you are in contact with people other than those you live with.
- Always check that the mask is well adjusted and covers both your nose and mouth.
- Warning: if you are sick this mask is not suitable. Ask your doctor for advice.
- This mask is not intended for medical workers or children under 3 years old.
- This mask does not replace social distancing measures (regular washing of hands, physical distancing, reduced contact with other people). It adds a physical barrier, to be used especially when you are in close contact with other people.

This mask is not a medical device in the sense of Reg EU/2017/745 (surgical masks) nor personal protective equipment in the sense of Reg EU/2016/425 (Masks type FFP2).

HOW TO WEAR THE MASK?

- BEFORE PUTTING MASK ON:
- 1 Wash the mask before wearing it.
 - 2 Before putting the mask on wash your hands with soap and water or a hydro-alcohol solution.
 - 3 Inspect the mask to ensure it doesn't have any holes, rips or degradations.
 - 4 It is recommended to wear the mask on bare skin, avoiding contact with hair.

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- 5 Never modify the mask in any way.

TO PUT MASK ON:

- 1 Hold the mask by the elastic straps.
- 2 Place it at the level of your mouth.
- 3 Place the elastic straps behind your ears and adjust the length thanks to the loops located on each bottom side of the mask.
- 4 Adjust the mask to cover your nose, mouth and chin.

WHEN WEARING MASK:

- 1 Avoid touching and moving it.
- 2 Never put it in position on the forehead or chin.

THE MASK MUST BE CHANGED:

- 1 When you have worn the mask 4 hours.
- 2 When you want to eat or drink.
- 3 When it becomes difficult to breathe.
- 4 If the mask gets wet.
- 5 If the mask is damaged.
- 6 If the mask is deformed and does not stay properly against your face.

TO REMOVE THE MASK:

- 1 Wash your hands with soap and water or a hydro-alcohol solution.
- 2 Unhook the elastic straps to peel off the mask from your face.
- 3 While waiting to wash the mask, isolate it in a plastic bag.
- 4 If the mask is distorted or worn out, throw it out right away in a trash can that closes.
- 5 Finally, wash your hands again with soap and water or a hydro-alcohol solution.

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be worn for a duration of 4 hours.

- 3 The safety of the mask is ensured thanks to Oeko-Tex certification Standard 100 of the textile materials used.

The performance of the masks was tested by the laboratories of IFTH and DGA Maltrise NRBC.

WARNING

Despite all the care taken in the manufacturing of these masks, it is the responsibility of the user, and in no case the manufacturer or supplier, to ensure the adequacy of the mask in the way it is used. Care should be taken to follow the recommendations and methods of use described above.

The Manufacturer is not responsible in the event of poor following of these mask usage recommendations and detailed by the AFNOR SPEC standard S76-001 "Barrier Masks" downloadable via the link

<https://telechargement-afnor.org/barrier-masks>

For Europe : CWA17553:2020.

CA6150, CA6190, CA6230, CA6210, CA6120 : 100% cotton

CA6140, CA6110 : external layers 100% cotton, internal layer 100% nylon

NUMBER OF MASK USAGES

Masks can be washed 50 times, each mask can therefore be used 50 times. To help you track and manage the count of your mask usages, please complete the following table by checking the boxes (table below): >> p.1

- 1 This mask has demonstrated more than 90% filtration effectiveness for 3 micron particles emitted by the person wearing the mask.
- 2 The breathability of the mask was measured at >96L.m-2.s-1 and allows it to